

Apogee Plus

Touch Free Intermittent Catheter System

Comfortable. Trusted quality.

Side-loaded catheter

helps reduce risk of urine backflow

Integrated 1500 mL

collection bag with tear strip

allows touch free catheterization

Smooth eyelets

aids comfortable insertion and removal

Protective tip

helps bypass bacteria in distal urethra

Pre-lubricated catheter

top half lubrication reduces friction during insertion



Portrait:

Bastien Perret

"It's not possible for anyone to live life and never have any problems. But for every problem, there is a solution. A person just has to go out and find it."



Apogee Plus

Touch Free Intermittent Catheter System

Touch Free Protective Tip

- Helps protect the catheter tip from exposure to bacteria in the initial 15 mm of the urethral opening
- Helps reduce the risk of carrying bacteria up into the urinary system
- Serves as a guide for proper and easy insertion

Side Loaded, Pre-Lubricated Catheter


- Pre-lubricated catheter (top-half) is designed to be easier to advance
- Side loaded catheter helps reduce risk of urine backflow
- Smooth catheter eyelets aid comfortable insertion and removal

1500 mL Urine Collection Bag with Tear Strip

- Allows secure, discreet catheterization; use anywhere, anytime
- Simple to empty the collection bag using the tear strip

Additional Trusted Features

- Protective cap covers the protective tip and helps prevent leakage after use
- Latex free **

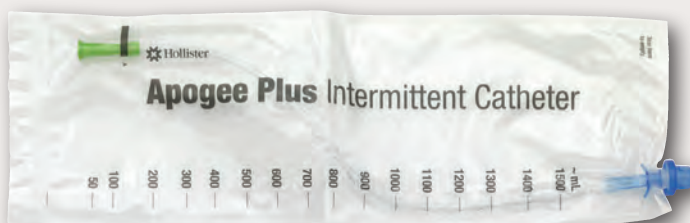

Apogee Plus Intermittent Catheter System, 16"

Apogee Plus	Apogee Plus Kit†	Size	Description
B6F	B6FB	6 Fr	Firm
B8F	B8FB	8 Fr	Firm
B10F	B10FB	10 Fr	Firm
B12F	B12FB	12 Fr	Firm
B14F	B14FB	14 Fr	Firm
B16F	B16FB	16 Fr	Firm
B18F	B18FB	18 Fr	Firm
B12C	B12CB	12 Fr	Coudé
B14C	B14CB	14 Fr	Coudé
B16C	B16CB	16 Fr	Coudé
B12S	B12SB	12 Fr	Soft
B14S	B14SB	14 Fr	Soft
B12R	B12RB	12 Fr	Red Rubber Latex*
B14R	B14RB	14 Fr	Red Rubber Latex*
B16R	B16RB	16 Fr	Red Rubber Latex*
B14F-Female	B14FB-Female	14 Fr, 6"	Firm

* Caution: This product contains natural rubber latex which may cause allergic reactions.

† Kit Includes: Apogee Plus intermittent catheter system, gloves, underpad drape, gauze, and PVP swabsticks

**Unless otherwise noted. Some versions of Apogee Plus intermittent catheters contain latex and are so described.

PRECAUTION:

Please consult a medical professional before using this product if any of the following conditions are present: Severed urethra, unexplained urethral bleeding, pronounced stricture, false passage, urethritis – inflammation of the urethra, prostatitis – inflammation of the prostate gland, epididymitis – inflammation of the epididymis (testicle tube). Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

WARNING:

To help reduce the potential risk of infection and/or other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

CAUTION:

Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare professional. Prior to use of this device, be sure to read (i) the complete information on how to use this device including Warnings, Cautions, and Instructions for Use, and (ii) all other package inserts and labels supplied with the product and accessories.

Rx Only. Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.



Hollister Incorporated

2000 Hollister Drive
Libertyville, IL USA

www.hollister.com

For detailed clinical questions
concerning our products:

1.888.740.8999

For orders only: 1.800.323.4060

For information visit

www.hollisterpeoplefirst.com

Hollister and logo, Apogee and Apogee IC
are trademarks of Hollister Incorporated.

©2013 Hollister Incorporated.
922326-713